

CASE STUDY

Lick Granuloma Treatment

K.LASER[™]_{USA}

1106 Harpeth Industrial Court

Franklin, TN 37064

866-595-7749 (v) • 615-261-3535 (f)

www.k-laserusa.com




OCTOBER 18



OCTOBER 21



OCTOBER 24

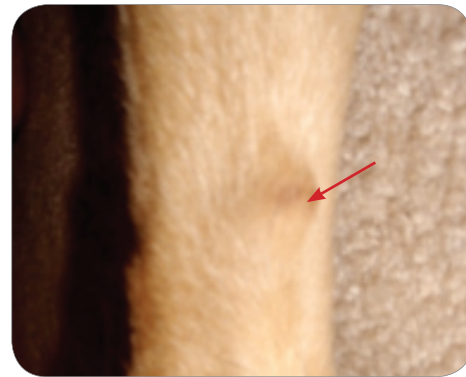

OCTOBER 28

Treatment Protocols:

MEDICINE:

Cephalexin 500 mg capsules (quantity 56)- Give 2 capsules by mouth every 12 hours until gone
Prednison 5 mg tablets (quantity 40)- Give 2 tablets twice daily for 5 days then give 2 tablets once daily.

LASER SETTINGS:

Superficial wound setting:

Phase 1: Power: 4.0 W

Wavelength: 800 nm

Phase Time: 60 sec.

Frequency: 5000 Hz

Phase 2: Power: 4.0 W

Wavelength: 800 nm

Phase Time: 60 sec.

Frequency: 7000 Hz

Phase 3: Power 4.0 W

Wavelength: 800 nm

Phase Time: 60 sec.

Frequency: 10000 Hz

TOTAL TREATMENT TIME: 3 min.

PRESET JOULES: 360.0

(First two treatments were done on a setting of 2.0 W for all phases)